

# Fixing and Maintenance Instructions

## Shower Chairs

**N.B. This equipment must be installed by a competent installer/ technician.**

Instructions for:

Milton Shower Chair	190Kg (30st.) weight limit
Luton Shower Chair	190Kg (30st.) weight limit
Bewl Plus Shower Chair	190Kg (30st.) weight limit
Deal Self-Propelled Shower Chair	190Kg (30st.) weight limit
Hythe Shower Chair	190Kg (30st.) weight limit
Charing Shower Chair	190Kg (30st.) weight limit
Darenth Shower Chair	190Kg (30st.) weight limit
Rochester Shower Stool	190Kg (30st.) weight limit
Dartford Shower Stool	190Kg (30st.) weight limit
Strood Shower Stool	159Kg (25st.) weight limit
Pembury Corner Shower Stool	159Kg (25st.) weight limit

**Do not** exceed the weight limited stated; doing so may put the user at risk.

### 1. All Shower Chairs/ Stools

Please examine your shower chair for any visible damage before using. If you see any damage or suspect a fault, please contact your supplier.

**In this case, please do not use your product as it may compromise your safety.**

### 2. Cleaning

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair.

Certain products have clip on/clip off backs and/ or seats, you can easily remove these if required clean with a mild detergent and soft cloth in a warm water solution. Do not exceed 50°C when cleaning the seat cover and/ or back as this will permanently damage the plastic.

N.B. Regular daily cleaning will reduce the build up of lime scale on your product.

Under No Circumstances use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for A.B.S, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.

### 3. Adjustments to Extension Legs

Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see *fig. 1*)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see *fig. 2*). **Never** install the extension legs unevenly or slanting, **this will compromise the safety of the user.**

## 4. Use of Bath Oils

Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance.

In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.

## 5. Reissue

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes;

P.A.T. test for electrical items, tightness of all nuts/ bolts/ castors and other screw in/ bolt in/ push in components. Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

