

# AIDAPT<sup>®</sup>

## INSTALLATION AND MAINTENANCE

## INSTRUCTIONS FOR QUIET KNIGHT

## MATTRESS STOPPER

Revision 4

Dated 29 May 2003



## **Fitting and Maintenance Instructions**

### **Quiet Knight Mattress Stopper**

The Quiet Knight Mattress Stopper is to be fitted in conjunction with the Quiet Knight Mattress Variator.

The purpose of the stopper is to reduce the possibility of ‘mattress creep’.

The fitting does not require any tools or specialist knowledge and can be carried out by any competent technician.

#### **1. To fit**

Firstly remove the mattress from the bed, exposing the base of the bed. Place the stopper with the U section resting onto the bed base at the centre (see *fig. 3*). The rectangular shape (see *fig. 4*) will be gripping the base and mattress when the mattress is replaced on the bed.

Fix the strap(s) provided from the stopper, looping around the stopper and lengthways up the bed, then loop around the swivelling section on the Quiet Knight Mattress Variator. Pull the strap as tight as possible and tie off, replace the mattress. The stopper is now ready for use.

Please note you will have to periodically tighten the stopper strap to maintain sufficient tension.

#### **2. Reissue**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes;


P.A.T. test for electrical items, tightness of all nuts/ bolts/ castors and other screw in/ bolt in/ push in components. Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

# Quiet Knight Stopper Fitting Instructions


*fig. 3*

## ELEVATION VIEW

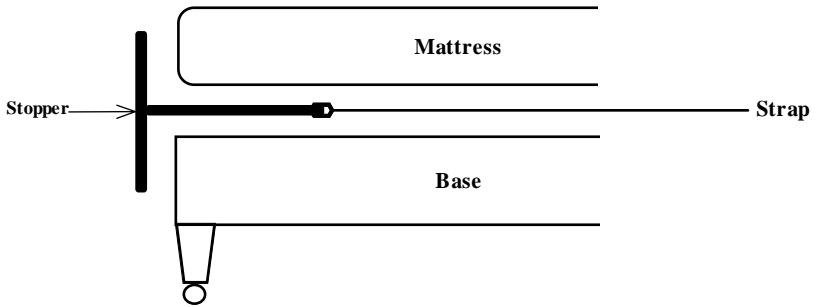

*fig. 4*

## FACE ON VIEW

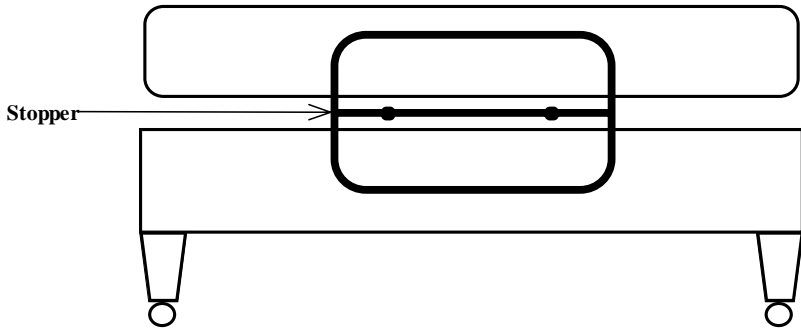

## **Important Information**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents and no warranty or representation concerning the information is given.

All products, names and titles including the word Aidapt and the Aidapt logo are registered trademarks of Aidapt Bathrooms Limited.

All information and text contained herein remains the property of Aidapt Bathrooms Limited and is protected by copyright.

Any reproduction of an information or text contained herein will be considered a breach of copyright.