

# Fitting instructions for side wedges.

## For use without side rails.

- Spread wipe clean draw sheet across the upper part of the mattress, contact strips up.
- Lift the mattress and pass the straps underneath to the opposite side of the bed.
- Clip into opposite buckle and tighten.
- Attach wedges to wipe clean sheet by hook and loop fastenings.
- Patients can be positioned and prevented from accidentally rolling out of bed.

## Product Care

The wedges and draw sheet can be wiped down with a mild detergent.

Spare drawsheets can be purchased separatly. 1 or 2 sets can be used and separate draw sheets are available on request. For further infomation on all other products in this range and costum made items, please contact us.

Thorpe Mill Ltd  
Parkwood St  
Keighley BD21 4PJ  
Tel : 01535 682 630  
Fax : 01535 682 639  
Email : [sales@thorpemill.co.uk](mailto:sales@thorpemill.co.uk)