

Cotside Bumper Information

Codes:- CSB, CSBH, CSBPB & TRCSB,



CSB 136 x 76 cm



CSBH 136 x 96 cm



CSBPB 200 x 76 cm



TRCSB 200 long

Fitting Instructions for Standard Cotside Bumper CSB CSBH & CSBPB

- Remove padding from outer bag.
- Taking one half of the pair, open flat.
- Lay padding over top rail and fasten under the bottom rail with Velcro fastenings.

Fitting Instructions for Cotside Bumper TRCSB

- Remove padding from outer bag.
- Taking one half of the pair, open flat.
- With the padding fully opened present padding to the top wooden rail from the inside of the bed. Wrap padding over top rail and Velcro to where the padding meets the net (underneath the bottom of top rail). For the second rail, wrap the padding round the underside of the second rail and Velcro to the top of the second rail.
- Repeat all steps for the other half of padding.

Please note when finished the closed Velcro's should be on the outside of the bed i.e. so the patient cannot easily release the padding.

Cleaning Instructions

These types of cotside bumpers can be wipe down with a mild detergent.