

# User Instructions

## Folding Easy Fit Bed Rail


### M15708

#### Maximum User Weight



#### General Safety Warning

Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.

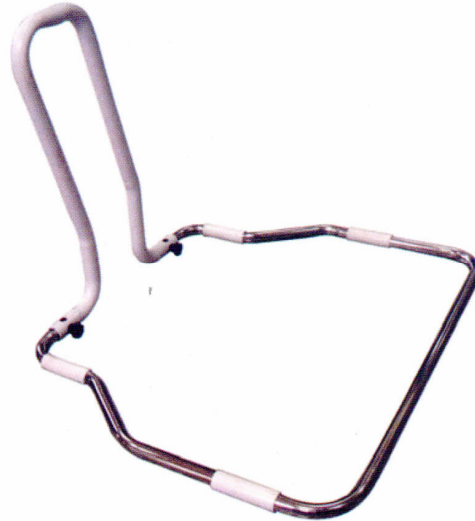

Shown assembled for right handed assist

#### Intended Use

The Folding easy Fit Bed Rail provides a stable support to help clients move themselves to a sitting position and assist them getting in and out of bed. This device is suitable for both divan and slatted domestic beds and is suitable for all bed sizes from single to king-size.

#### Safety Precautions

This device is suitable for adult use only.  
It is not suitable for use with young children.  
This device is not designed for use with profiling beds.  
Please ensure the user does not exceed 127kg (20 stones)  
The position of the device should be such that when placed under the mattress with the handle fitted, the client should be able to reach the handle without stretching.

#### Assembly

Note: It is intended that the white handle should be positioned so that it angles away from the bed head as shown in the image above.

Unscrew the two knobs and rotate the white handle to the desired side, depending on whether it is on the right or left hand side of the bed.  
Replace the knob (from side)

Insert bed rail fully under mattress, with angle pointing away from bed head and positioned so that the client can reach without stretching.

#### Cleaning Instructions

Clean regularly, wiping down with a mild detergent or disinfectant solution.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.



Shown on slatted bed for right hand assist

Manufactured for Nottingham Rehab Supplies  
Clinitron House, Excelsior Road, Ashby Park  
Ashby de la Zouch, Leicestershire, LE65 1JG  
United Kingdom

Tel: 0845 120 4522

Int'l: +44 (0) 1530 418378

Fax: 0845 121 8112

Email: [customerservice@nrs-uk.co.uk](mailto:customerservice@nrs-uk.co.uk)

This product is from a range of over 3000 daily living aids available from Nottingham Rehab Supplies. To view the full range visit [www.nrs-uk.co.uk](http://www.nrs-uk.co.uk) or request a catalogue from the above number.