

User Instructions

Folding Easy Fit Bed Rail

M15708

Maximum User Weight:



General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The Folding Easy Fit Bed Rail provides a stable support to help clients move themselves to a sitting position and assist them getting in and out of bed. This device is suitable for both divan and slatted domestic beds and is suitable for all bed sizes from single to king-size.

Safety Precautions

- This device is suitable for adult use only.
- It is not suitable for use with young children.
- It is not designed for use with profiling beds.
- Please ensure the user does not exceed the maximum user weight: 127kg (20 stones)
- Ensure the mattress is resting firmly against the bed rail handle

Important safety note: There must be a gap of at least 250mm from the headboard to the Bed Rail Handle, and the mattress should rest against the handle of the rail when fitted to reduce the risk of head/limb entrapment.



Diagram 1

Showing Bed Rail fitted on a slatted bed for a right hand assist

Installation

The position of the Bed Rail (which should ideally be based on a clinical assessment) should be such that when placed under the mattress with the handle fitted, the client (from a lying position) should be able to reach the handle without stretching.

It is intended that the handle should be positioned so that it angles away from the bed head as shown in *Diagram 1*.

Assembly

- Unscrew the two knobs (A), as shown in *Diagram 3* and remove the bolts (C) and square location spacers (B).
- Rotate the white handle to the desired side, depending on whether it is on the right or left hand side of the bed, as shown in *Diagram 2*.
- Replace the bolts and spacers and tighten the assembly with the knobs, such that the knobs are fitted on the inside of the frame, see *Diagram 2*.
- Insert bed rail fully under mattress, with angle pointing away from bed head and positioned so that the client can reach without stretching, as previously described.



Diagram 2
Showing Bed Rail assembled for Left Hand Assist

IMPORTANT NOTE:

If the Bed Rail is to be used for transfers onto a divan bed, for example from a wheelchair, it is recommended that two bed rail straps, looped around the frame and divan base are used to provide extra security.

The strap part number is, L75930 and can be ordered from NRS Customer services – details shown below.

Care & Maintenance

- Inspect at regular intervals for wear and damage.
- Remove from use immediately if there are signs of wear, damage or the handle is excessively loose.
- Check there is no gap between the rail handle and the mattress
- Inspect regularly to ensure the Hand Nuts (A) are fully tightened
- If a bed rail strap has been fitted for extra security check regularly that it is tight and the rail is correctly positioned

Cleaning Instructions

Clean regularly, use a general purpose, neutral detergent and warm water. Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

To disinfect; wipe with a mild chloride disinfectant (solution of 1000 parts per million).

This product is also suitable for industrial cleaning up to 80°C

After cleaning ensure the product is dried thoroughly.

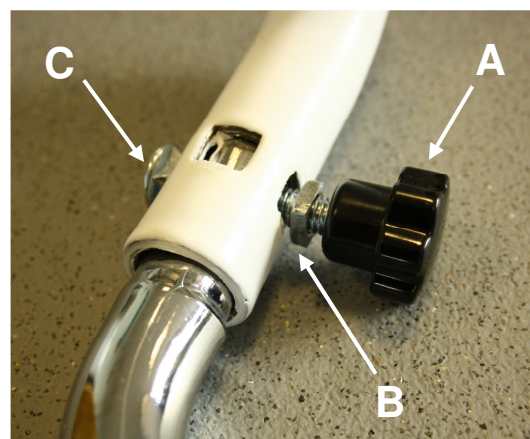


Diagram 3
Showing Detail of fixing

Manufactured for
Nottingham Rehab Supplies
Clinitron House, Excelsior Road,
Ashby Park, Ashby de la Zouch,
Leicestershire, LE65 1JG
United Kingdom

Tel: 0845 120 4522
Int'l: +44 (0) 1530 418378
Fax: 0845 121 8112
Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from Nottingham Rehab Supplies. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.

www.nrs-uk.co.uk

