

# User Instructions

## NRS EasyFit Bed Rail

| M76333 | With Straps | M76321 | Without Straps |
|--------|-------------|--------|----------------|
|--------|-------------|--------|----------------|

### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### 1.0 Intended Use

The EasyFit Bed Rail provides stable support to assist clients getting into and out of bed. When in bed the rail can be used to aide users rising to a sitting position or repositioning when lying.

Designed for use with divan beds, most slatted domestic beds and is suitable for all bed sizes from single to king-size.

### 2.0 Safety Precautions

**WARNING!** To avoid possible head entrapment, position the Bed Rail at least 320mm (12½") away from both headboard and footboard. See figure 3.

Always ensure that the mattress is as close as possible to the Bed Rail. Any gap could allow the client to become wedged in-between.

Do not exceed the product maximum user weight 190kg, 30st.

Do not fit this product to profiling beds.

Not suitable for use by children.

### 3.0 Unpacking

Remove packaging and check all parts carefully. If any parts appear damaged, do not assemble and contact your supplier immediately.

**Contents:**

- Bed Rail Handle X 1
- Bed Rail Base X 1
- E-Clip X 2
- Strap (Product M76333 only) X 2

### 4.0 Assembly

Slot both ends of the Bed Rail Base into the handle and secure using the 'e-clips', figure 2.

### Maximum User Weight



Figure 1 – EasyFit Bed

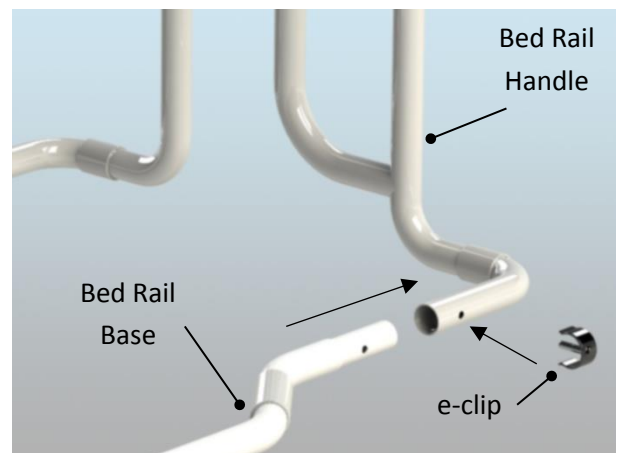


Figure 2 - Assembly

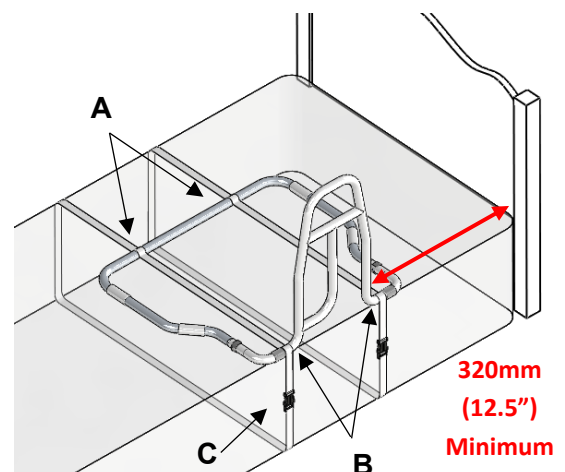


Figure 3 – Fitted to Divan Bed

## 5.0 Installation

Remove the mattress and place the Bed Rail onto the base of the bed.

Position the Bed Rail such that the user can reach the handle from a relaxed lying position without having to stretch. *For the safety of the user this must be at least 320mm away from the headboard or footboard.*

If no straps are being used replace the mattress.

### **To secure with straps – Divan Bed:**

Take the first strap. Loop the female buckle over and under the Bed Rail base at points A and B, figure 3.

Pull the strap to allow the buckle to hang down the side of the bed.

Take the other end of the strap and pass it under the divan base.

Fasten the buckle and pull the strap tight such that it wraps tight around the base of the bed.

Repeat for the second strap.

### **To secure with straps – Slatted Bed:**

*Please note: To use this method the bed slats must be securely fixed in place.*

Take the first strap. Loop the female buckle over and under the handrail base and a single slat at point A, figure 4.

Pull the strap to allow the female buckle to pass under the slats then up and around the handrail base at point B, figure 4.

Wrap it around a single slat at point B.

Pass the strap back under the slats to fasten the buckle.

Pull to tighten the strap and securely position the bed rail.

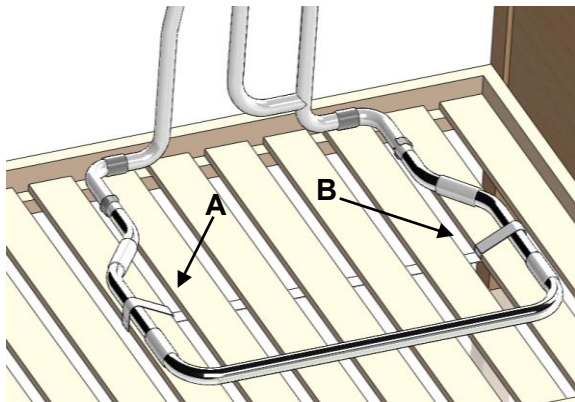


Figure 4 – Fitted to Slatted Bed

## 6.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Allow the product to dry fully prior to use/storage.

## 7.0 Maintenance

Check that all fixing clips are inserted correctly and that there is no excessive movement.

Check that straps are tight and the bed rail is correctly positioned.

Inspect the product regularly for signs of wear or damage.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 8.0 Accessories

L75930 Fixing Strap

M84780 Slip Resistant Netting (white)

## 9.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

## 10.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

### **NRS Healthcare**

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK  
Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)



**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**