


PRODUCT INFORMATION SHEET

Novara Rise Easy - Single

These instructions cover the 3'0" Bed Range

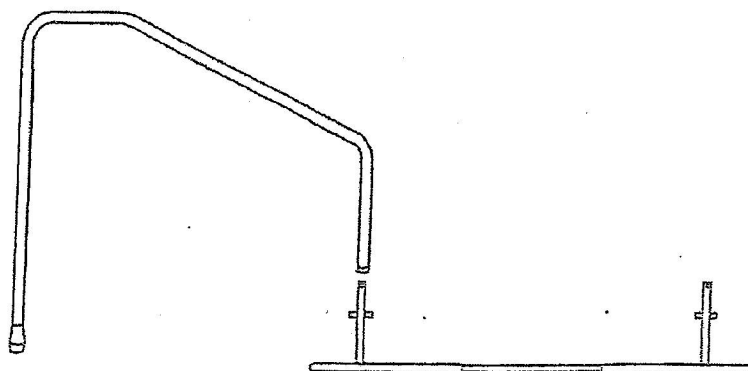

1. Remove all parts from the box and lay parts on the floor.
2. Push both silver coloured connecting rods into both tubes on base plates to bed size required. **DO NOT PULL TUBE PAST THE WARNING DIMPLE!**
3. Slide assembly under the head section of the bed and align base plate to bed castors.
4. Lift bed and slide assembly under castors of the bed, lower bed onto base plate ensuring that the castors sit onto the base plate.

PLEASE NOTE: Some beds are very heavy get help if required.

5. Slide handrail over the location tube and push down until it hits the location stop pin. Please note the rail can be fitted to the left or right side of the bed.

