

Savanah Slatted Bath Board:

AA1091A, B , C and D


Savanah Slatted Shower Board:

AA1093

Handle:

AA1092

Maximum user weight

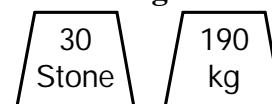


Please keep these instructions for future reference.

INTENDED USES

- To compensate for a disability by providing a sitting surface for a person taking a bath or shower



A

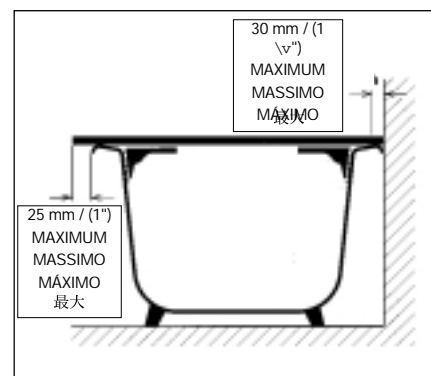

B

CAUTIONS

- The bath or shower board is not to be used to support more than 190 kg / 420 lb. / 30 st.
- Both ends of the bath or shower board must be supported securely by the sides of the bath without an excessive overhang.
- Be sure the bath or shower board is securely in place and the brackets are snug against the sides of the bath before putting weight on the board.
- Do not stand on the board.
- Use of a slip-resistant bath mat is strongly recommended.
- Cover the bath or shower board with a towel to help reduce slipperiness, if required.

INSTRUCTIONS FOR INSTALLATION

1. Remove all components from the carton and snap the two brackets together as shown in A.
2. Using the wing nuts, fasten the brackets loosely to the underside of the board and slide them as far toward the centre of the board as possible.
3. Place the board on the bath, being sure of the following:

- The board must rest securely on both sides of the bath with an overhang that does not exceed 25 mm (1") (B).
 - Each end of the board must be supported by the rim of the bath at a point at least 30mm (1 1/4") in from the end of the board (B) (see **Specifications** for the width suitability of each board).
 - The bath must have an edge wide enough to support the ends of the board.
 - The board must be fitted squarely across the bath - **do not position it at an angle.**
4. Move the brackets toward the sides of the bath until they are snug against the sides.
 5. Tighten the wing nuts securely.
 6. Repeat the fitting procedure each time the bath or shower board is placed on the bath.
 7. The bath or shower board can be used with a bath seat that enables the user to sit lower in the bath.

Handle

The handle can be used to assist in transfers onto and off of the board.

8. To attach the handle, position it over the holes in the butterfly-shaped cross brace on one end of the bath or shower board. Fasten it into position with the screws and key provided.
9. The curve of the handle can be positioned according to personal preference.

SPECIFICATIONS

- AA1091A: 660 mm (26") long; adjustment to suit widths inside of the bath from 483 mm (19") to 584 mm (23"). 232mm (9 1/4") wide.
- AA1091B: 711 mm (28") long; adjustment to suit widths inside of the bath from 533 mm (21") to 635 mm (25"). 232mm (9 1/4") wide.
- AA1091C: 762 mm (30") long; adjustment to suit widths inside of the bath from 584 mm (23") to 686 mm (27"). 232mm (9 1/4") wide.
- AA1091D: 686mm (27") long; adjustment to suit widths inside of the bath from 508mm (20") to 610mm (24"). 232mm (9 1/4") wide.
- AA1093: 686mm (27") long; adjustment to suit widths inside of the bath from 508mm (20") to 610mm (24"). 356mm (14") wide.

CARE AND CLEANING

All parts can be cleaned with a household disinfectant or mild detergent and water.