

User Guide SLIDE SHEETS

The slide sheets are made in colours to indicate their size and have a contrasting loop for hanging

The sheets are in the form of a tube with very low friction inner surfaces.

They are used for various positioning jobs like moving the patient up or down the bed or simply turning. Along with many others.

The two inner surfaces slide easily against one another avoiding any shear force to the patient

Instructions for use

First lay the sheet close to the heaviest parts of the body, hips/shoulders, then roll him/her onto the sheet.

Now pull on the upper layer in the direction required.

The sheet can be removed by gently easing out the bottom layer of the sheet.

Care instructions

If the slide sheet needs washing please see below.




40° wash



Low
tumble dry



Do not dry
clean

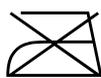

Do not iron



Do not bleach

Pack contains:- One

Small
70x70 cm

Large
150x70 cm

Medium
120x70 cm

Wide
125x100

Extra Long
200x70 cm

Thorpe Mill Ltd
Unit 1a Aireside Business Park
Royd Ings Ave
Keighley
West Yorkshire BD21 4BZ
T:01535 682 630
F:01535 682 639
E: sales@thorpemill.co.uk

User Guide SLIDE SHEETS

The slide sheets are made in colours to indicate their size and have a contrasting loop for hanging

The sheets are in the form of a tube with very low friction inner surfaces.

They are used for various positioning jobs like moving the patient up or down the bed or simply turning. Along with many others.

The two inner surfaces slide easily against one another avoiding any shear force to the patient

Instructions for use

First lay the sheet close to the heaviest parts of the body, hips/shoulders, then roll him/her onto the sheet.

Now pull on the upper layer in the direction required.

The sheet can be removed by gently easing out the bottom layer of the sheet.

Care instructions

If the slide sheet needs washing please see below.



40° wash



Low
tumble dry



Do not dry
clean



Do not iron



Do not bleach

Pack contains:- One

Small
70x70 cm

Large
150x70 cm

Medium
120x70 cm

Wide
125x100

Extra Long
200x70 cm

Thorpe Mill Ltd
Unit 1a Aireside Business Park
Royd Ings Ave
Keighley
West Yorkshire BD21 4BZ
T:01535 682 630
F:01535 682 639
E: sales@thorpemill.co.uk