

# Product Technical Data Sheet 051

## Atlas Advance Turning/Transfer Aid

### Product Range

P11437 Atlas Advance Transfer Turning Aid

### Product Description & Intended Purpose

The transfer aid is designed to provide assistance to a user when rising from and lowering to a seated position.

It can also be used indoors to complete short distance transfers and for easy manoeuvring of users in confined spaces.



Users must be able to support their own weight and maintain balance for a reasonable time before using this product.

### Product Features

- Easy to assemble with no tools required.
- 4-bar handle for users of different heights.
- Will fit under most divan beds.
- Twin brake levers (locking both sides simultaneously).
- Non-slip mat on platform.
- Height & width adjustable padded knee supports.
- Plastic bumpers to protect furniture.
- Optional Safety Belt (S,M,L) with two fixing heights.
- Durable construction with powder coated steel frame.



### TECHNICAL SPECIFICATION - ALL DIMENSIONS IN MM (INCHES)

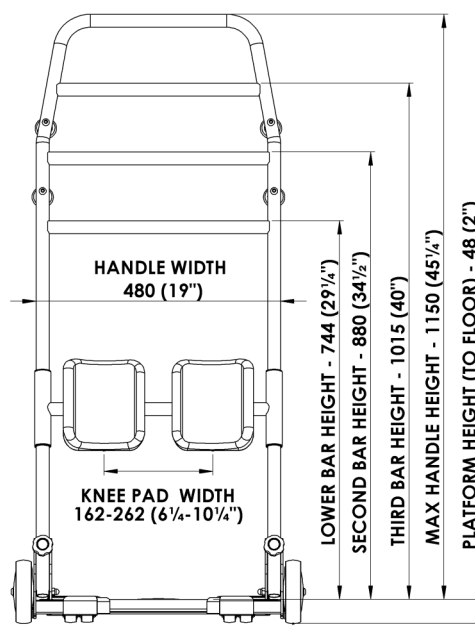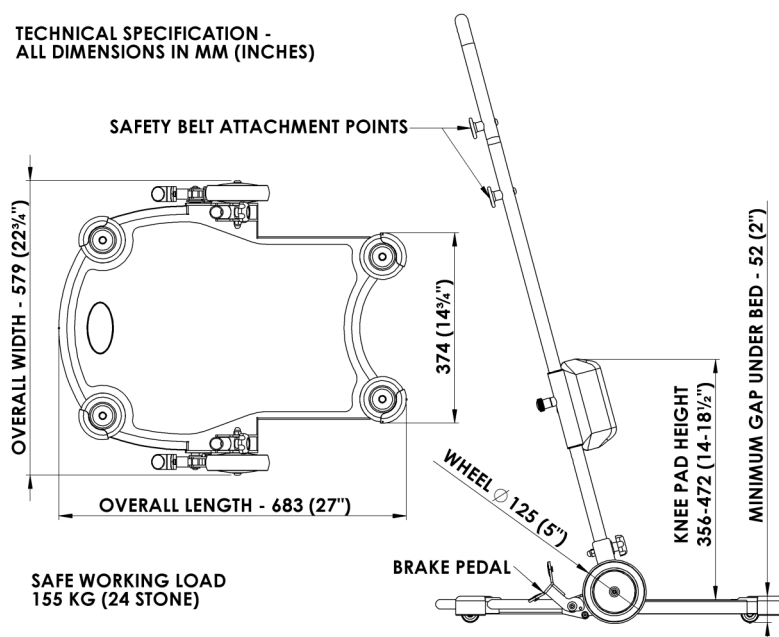

## PTDS051

### Atlas Advance Turning/Transfer Aid

#### Installation Information: *(Note: for detailed information refer to User Instructions)*

- Product is delivered in 2 parts; handle and standing platform.
- To fit the handle, slot the handle into position and secure using the 2 hand-wheels provided.

#### Safety Precautions

- Do not exceed the maximum user weight 24st, 155kg.
- Do not leave the user unattended on the product.
- Not for use by children.
- Do not use product outdoors. Only use on indoor level surfaces and not over thresholds >10mm.
- Always engage the brakes when transferring a user from seated to standing or vice versa.
- Plan route prior to use and take care to avoid obstacles when transporting a user.
- NRS recommend the user wears the optional safety belt at all times when transporting a user.

#### Cleaning

- To clean the Transfer Aid, use a general purpose, neutral detergent and warm water.
- Ensure the product is fully dried prior to use/storage.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- Do not use abrasive cleaners or cloths as these could permanently damage the surface.
- To clean the belt, hand wash at a low temperature using a mild detergent.
- Do not machine wash the safety belt or use bleach on the belt.

#### Care & Maintenance

- Check the castors are securely fitted and are free to rotate, remove any build-up of dirt/debris.
- Check the brakes are operational and that there are no signs of wear.
- Check the adjustment of the leg support bracket runs smoothly and securely locks in position.
- Store the Transfer Aid on a flat surface with the brakes locked.

#### Accessories / Spares

- Optional Safety Belt available in 3 sizes, Small (P11449), Medium (P11450), Large (P11462).
- Optional Knee Pad Covers P18808.
- Full set of spares available to enable the product to be repaired / refurbished:  
Knee Pads (P11539), Set of Castors (P11565), Pair of Wheels (P11577), Brake Assembly (P11589),  
Plastic Bumpers (P11553), Pair of Harness Fixing Knobs (P11541), Handle Hand-wheels (11590)

#### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville,  
Leicestershire. LE67 1UB UK