



INSTALLATION AND MAINTENANCE
INSTRUCTIONS FOR PERCHING STOOLS

Revision 4

Dated 29 May 2003



Fixing and Maintenance Instructions

Perching Stools

N.B. This equipment must be installed by a competent installer/ technician.

Instructions for:

Malling Perching Stool	154Kg (25st.) weight limit
Astral Perching Stool	154Kg (25st.) weight limit

Do not exceed the weight limits stated, doing so will put the user at **risk.**

1. Assembly

Your perching stool is delivered complete and does not require assembly.

2. All perching stools

Please carefully examine your perching stool for any visible damage before using. If you see any damage or suspect a fault, please contact your supplier.

In this case, please do not use your perching stool as you may compromise the safety of the user.

3. Adjustable Height

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see *fig. 1*)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see *fig. 2*)



Note the perching stool is designed to slant to the front. **Do not** adjust the legs to make the seat even as **this will compromise the safety of the user.**

Perching Stools continued...

4. Cleaning

a. Malling Perching Stools

It is important that you only clean the upholstery using a **non-abrasive cleaner** or **mild detergent** with a **soft cloth**. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair. Always ensure that you wipe dry the equipment after cleaning.

The products in this group can be treated with formaldehyde (upholstery only), but it is important that this treatment be carried out by an approved contractor or an experienced person and under suitable conditions.



On **No Account** must you attempt to clean with this method if you on your premises do not meet the legal requirements and guidelines governing the use of hazardous chemicals.

b. Astral Perching Stools

Astral perching stools have contoured polyurethane foam seats and backs which have been designed to clip on/ clip off, this is to ease cleaning and service.

To clean, carefully remove the seat or back by applying pressure to the rear, either side of the clips. Clean as upholstered Malling perching stools. To reaffix, simply align the clips on the underside of the seat and or back and apply pressure until you feel the seat and/ or back clip on. Test both the seat and the back for stability before use or issue.



Caution. Please do not lift Astral Stools by the padded seat or back as they may possibly come loose. Always lift your Astral Stool by the frame.

5.

Composition

The perching stools are made from mild steel PPA/ epoxy coated with upholstered CMHR foam and fire retardant vinyl seats or contoured polyurethane seats.

6. Reissue


If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes;

P.A.T. test for electrical items, tightness of all nuts/ bolts/ castors and other screw in/ bolt in/ push in components. Also check all upholstery for security, splits, exposed staples, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

Figures and Diagrams

Extension leg height adjustment Instructions

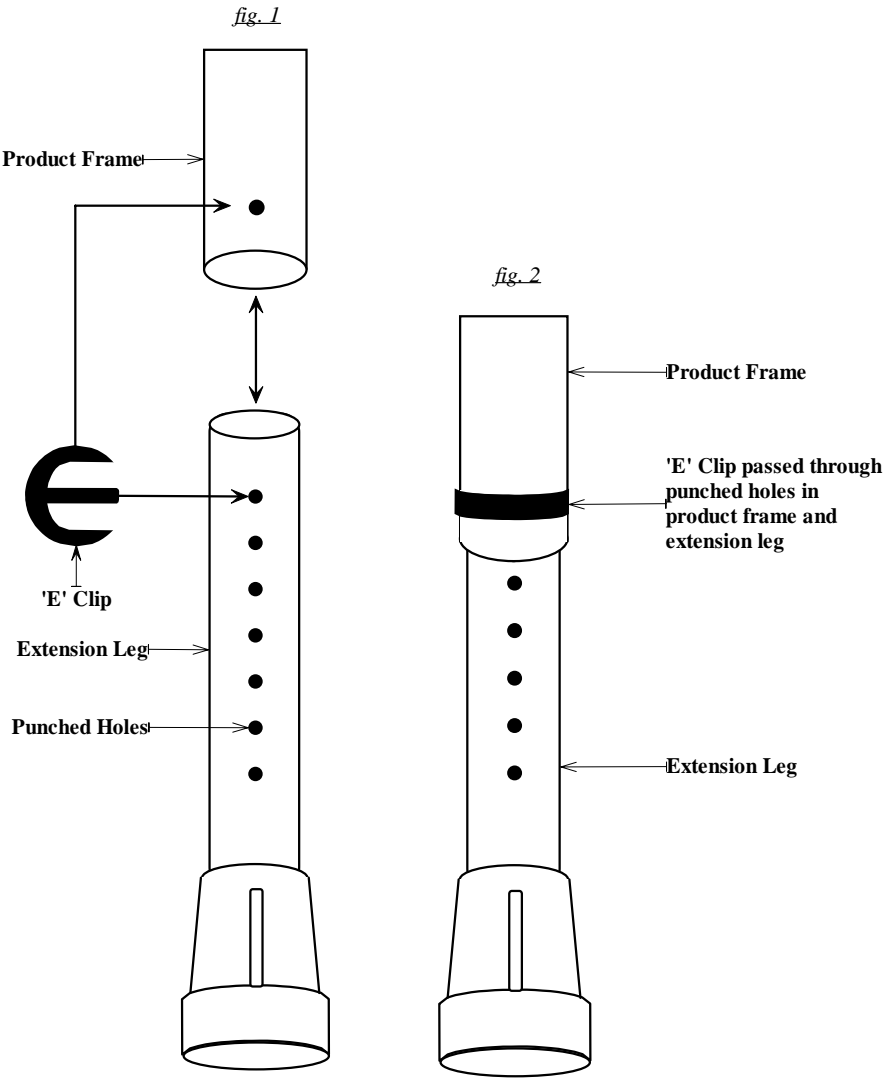

Important Information

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents and no warranty or representation concerning the information is given.

All products, names and titles including the word Aidapt and the Aidapt logo are registered trademarks of Aidapt Bathrooms Limited.

All information and text contained herein remains the property of Aidapt Bathrooms Limited and is protected by copyright.

Any reproduction of an information or text contained herein will be considered a breach of copyright.