

User Instructions

Mowbray Lite Toilet Frame & Seat Range

M66613 - Mowbray Lite - Width Adjustable - Flat Pack

M66625 - Mowbray Lite - Fixed Width Assembled

M92740 - Mowbray Lite 510 - Fixed Width Assembled

General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Maximum
User
Weight



Intended Purpose

The Mowbray Lite toilet frame and seat range is designed to assist those who have difficulty in lowering onto or rising from a standard toilet. The plastic moulded seat offers support in a natural position and the frame is height and width* adjustable.

(*Only available with M66613 – Mowbray Lite Flat Pack)

Unpacking

- Remove all items from the packaging.
- Check all component parts are present (see contents list overleaf)
- Check for any sign of damage.
- If you see any damage, or suspect a fault, do not assemble the product; contact your supplier immediately.



Fitting Instructions

1 - Mowbray Lite Flat Pack only: Assemble the frame sides to the centre bars, slide until the desired width is achieved, align the holes and secure using the four E clips provided. Please ensure the equivalent holes are used on each side of the frame to ensure it remains square.

2 - All Models: Remove the moulded seat from its polythene bag and snap into place on the centre bars of the frame.

3 - Ensure the seat is firmly in place and secure at both back and front.

4 - Adjust to the required height by removing each Eclip in turn.

5 - Slide each leg up or down to the required height. Align the holes in and replace Eclip.

Contents M66613 Flat Pack	
Description	Qty.
Toilet seat	1
Frame Side LH	1
Frame Side RH	1
Frame Centre bar	2
Leg assembly	4
E-clips	8

Note: M66625 & M92740 are delivered assembled.

Safety Precautions

Please ensure the user does not exceed the maximum user weight 190kg (30 stones).

6 - Ensure that all leg extensions are set at the same height to prevent rocking.

7 - Place the Mowbray over the toilet with the cutaway of the seat at the front and the integral splash guard placed centrally over the toilet bowl.


8 - When not in use Mowbray[®] can be stacked with the seat in situ. It is recommended that no more than five frames are stacked together for long term storage

Fitting Instructions Floor Fixing Kit

(If a permanent fix is required a Floor Fixing Kit is available as an accessory . see spare & accessories table.

1. Adjust frame to desired height.
2. A total of 12 screws must be used to fix the frame to the floor, using fixing methods appropriate to the floor's construction. - see diagram below

Please Note: Height adjustment is not possible once the legs are secured to the floor

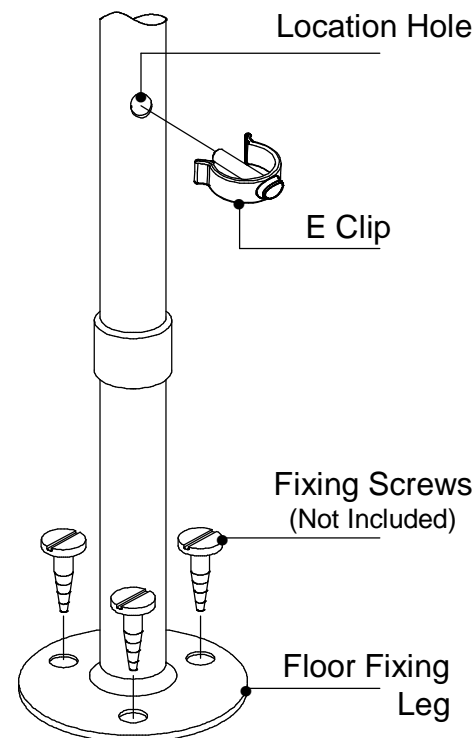

Care and Use

1. The seat may be removed from the frame and autoclaved on cycles up to 70°C.
2. The frame may be cleaned using a non-abrasive cream cleaner or detergent solution.
3. Check the ferrules on the freestanding versions monthly for signs of wear.
4. Check floor fixing screws weekly.
5. Check E clips every 2 months.
6. Maximum user weight is 190kg (30st).

Spares & Accessories	
F11572	Floor Fixing Kit (Set of 4)
F17938	Ferrule
M66820	Seat

IMPORTANT

For information on how to obtain the User Instructions in other languages and formats for use by people with visual, reading or cognitive impairments, please contact: info@nrs-uk.co.uk or telephone 0845 120 4522.

Additional copies of this user instruction are available on our website: www.nrs-uk.co.uk.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardonia Hill,
Coalville, Leicestershire.
LE67 1UB UK

Tel. Main Office: 0330 100 00 98
Tel. Customer Services: 0845 120 4522
Email: customerservice@nrs-uk.co.uk

These products are from a range of over 3000 daily living aids available from NRS Healthcare. To view the full range visit :

www.nrs-uk.co.uk or request a catalogue from the above number.



www.nrs-uk.co.uk