


# User Instruction for the Rear Loading Bariatric Commode Adjustable Height & Fixed Arms X221

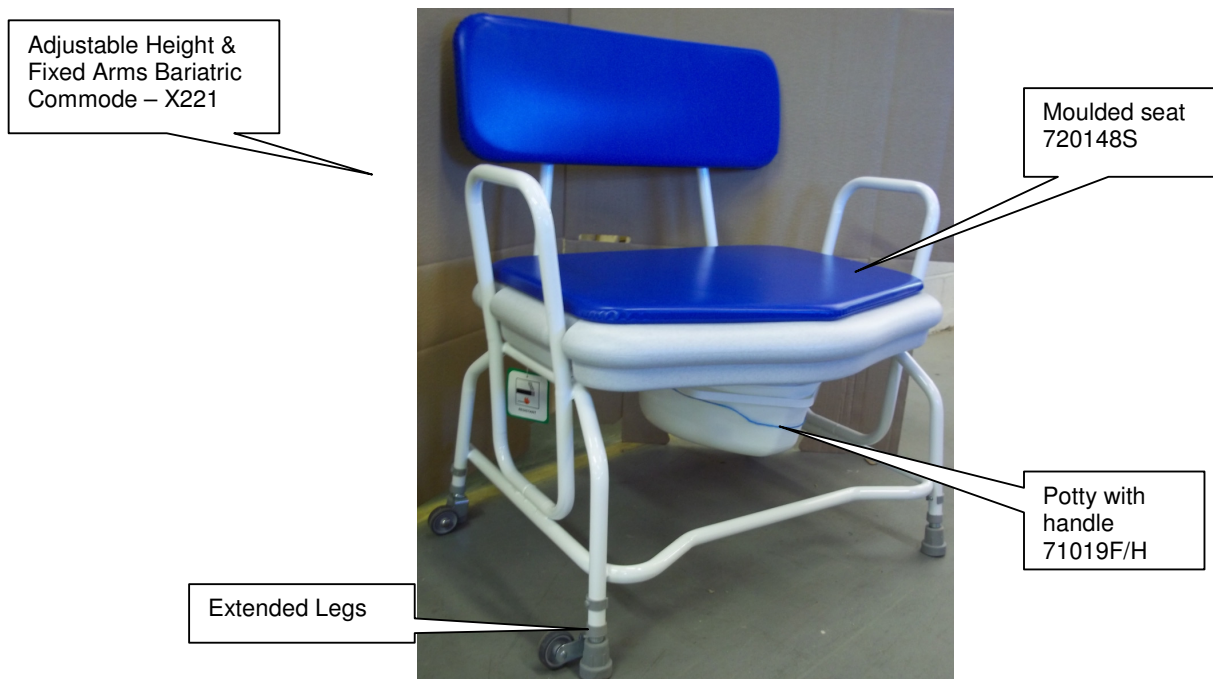

## 1. Introduction

The Bariatric Commode has a steel frame and is fitted with transportation wheels for ease of use. The frame has been coated with a nylon modified epoxy powder for ease of cleaning. The padded upholstery is heat-sealed so it can be cleaned without risk of water ingress. On this version the height can be adjusted. (Please note the transport wheels are designed to aid the movement of the product, **this is not a mobile commode!**)

Product Dimensions	
Seat height (from-to)	450mm-580mm
Padded seat thickness	20mm
Width between arms	735mm
Seat to top of arms	200mm
Seat to top of back	405mm
Orifice width	255mm
Orifice depth	300mm
Maximum footprint	800mm x 540mm

## 2. Contents of the Box

- Extra Wide Commode (X221)
- Potty with handle (71019F/H)
- Moulded Seat (720148S)
- Vinyl Seat
- Extended Legs & Pin Clips
- User Instructions (this leaflet)

## 3. Assembly Instructions


- Carefully unpack and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

## 4. Correct Use

**Transportation:** To transport the commode, using the armrest, tilt the commode upwards so that the transportation wheels come into contact with the floor – it is usually easier to pull rather than to push the commode.

### Adjustable Height:

- Remove the pin clips from all leg uprights.

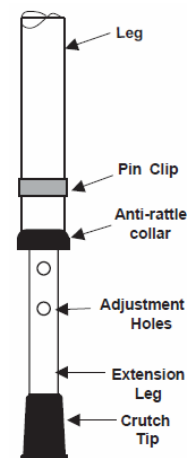

## **User Instruction for the Rear Loading Bariatric Commode Adjustable Height & Fixed Arms X221**

- Select the desired height of the frame by gently sliding out each extension leg from the frame (NOTE – this is a telescopic action).
- When desired height is obtained, refit the pin clip into all of the legs making sure pin clip goes through each leg and its extension leg. The frame is now ready for use.

### 71019F/H Potty:

This product was developed for greater ease of use. The rear handle was designed intended to allow easy fitting and removal from the commode and greater control when carrying and emptying. The upper handle when put in the upright position locks into position for stability. To fold it down, push the handle vertically downwards to release the locking tabs before folding it backwards towards the rear handle.

- To fit the pan (X221), ensure that the pan handles are lifted clear of the support brackets then slide the pan across the rear supporting brackets fitted to the commode until contact is made with the front of the bracket.
- To remove the pan, ensure the handles are clear of the support brackets and then slide the pan out from the rear of the commode.
- The upper handle will require to be folded down before fitting to the commode.



Handle folded down



Handle raised for carrying

### **5. Cleaning**

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.

#### Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependant on the manufacturer's instructions.

### **6. Maintenance Requirements**

The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

### **7. Warning**

- This product is intended for users of a maximum of 250kg/40stone in weight.
- We recommend a full risk assessment should be carried out before use.

### **8. Warranty**

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we have conducted simulated testing in accordance to the specified maximum weight limit and relevant standards and foresee that the product has an intended life of 3 years continuous use.

### **9. Disposal Instructions**

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

### **10. Contact Details**

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare, Cefndy Road, Rhyl, Denbighshire, LL18 2HG

Tel: 0044 1745 343877 (Quality Assurance Dept)

E-mail: [cefndyengineering@denbighshire.gov.uk](mailto:cefndyengineering@denbighshire.gov.uk)